PRONTEX[®]

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Wind

**SISTEMA COMPLETO
PER AEROSOLTERAPIA
COMPRESSOR NEBULIZER**

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Carefully read this manual before use and keep it with the device.

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EN

GENERAL WARNING.....	15
IMPORTANT SAFETY RULES	15 - 16
TECHNICAL CHARACTERISTICS	16
SYMBOLGY	18
MAINTENANCE	18 - 19
CLEANING DEVICE	19
CLEANING AND WASHING OF ACCESSORIES	19
STANDARD ACCESSORIES	20
INSTRUCTION FOR USE	21 - 22
INSTRUCTION FOR USE - NASAL IRRIGATOR	22
CLEANING THE DEVICE - NASAL IRRIGATOR	23
MAINTENANCE - NASAL IRRIGATOR	23
RISK OF ELECTROMAGNETIC INTERFERENCE AND POSSIBLE REMEDIES	24 - 25
RULES FOR RETURNING AND RAPAIRING	26

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PRONTEX WIND is an aerosol therapy system, electrically powered 230V ~ / 50Hz, equipped with a highly effective nebulizer (featuring 4 different useful positions to adjust the nebulization speed), intended for nebulization of aerosolized drug administration to support respiratory tract treatment.

The medical device is designed for continuous use. The high thermal insulated plastic body complies with European Safety Rules. Device equipped with carrying bag, for transport and maintain the apparatus and all supplied accessories.

Indications for use: Device used on Home-Care environment indicated for supporting the aerosolized administration of drugs for the treatment of airway pathologies.

Expected Patient Group: A device suitable for use in the treatment of patients of all ages (patients without cognitive deficits), when inhalation therapy is short e.g. 7-10 days and especially when there is also a need to thin the secretions.

Expected users group: Medical device intended for use in the home-care environment are used by lay users. The use of the devices by children and dependent persons must always be envisaged under the close observation of an adult in full mental capacity. No special training is required but the user manual contains some points for its correct use.

GENERAL WARNING

READ INSTRUCTION MANUAL CAREFULLY BEFORE USE

DRUG ADMINISTRATION MUST BE UNDER MEDICAL CONTROL

THE INSTRUMENT MUST NOT BE DISASSEMBLED, FOR A TECHNICAL SERVICE ALWAYS CONTACT SAFETY S.p.A

DEVICE EQUIPPED WITH CARRYING BAG. REMOVE THE DEVICE FROM CARRYING BAG BEFORE USE

IMPORTANT SAFETY RULES

1. On opening the packaging, check the integrity of the appliance, paying particular attention to the presence of damage to the plastic parts, which may make access possible to internal live parts and also to breakage and / or peeling of the power supply cable. **In these cases don't connect the plug to the electric socket. Carry out these controls before each use;**
2. Before connecting the appliance always check that the electric data indicated on the data label and the type of plug used, correspond to those of the mains electricity to which it's to be connected;
3. Never leave the appliance inserted if not necessary disconnect the plug from the mains power supply when it is not being used;
4. Respect the safety regulations indicated for electrical appliances and particularly:
 - Only use original accessories and components provided by the manufacturer to guarantee the highest efficiency and safety of the device;
 - Never immerge the appliance into water;
 - Position the appliance on flat stable surfaces;
 - Position the device in a way that the air inlets on the back aren't obstructed;
 - Never use the device in environments which have anaesthetic mixtures inflammable with air, oxygen or nitric oxide;
 - Don't touch the device with wet hands and always prevent the appliance coming into contact with liquids;
 - The use of this device by children and / or incompetent person always requires the careful surveillance of an adult in possession of their full mental faculties;
 - The medical device, and most of all the nebulae, must be kept out of children's reach as it contains small parts that could be swallowed;
 - Don't leave the appliance connected to the power supply socket when not in use;
 - Don't pull the power supply cable to disconnect the plug remove the plug from the mains socket correctly;
 - Store and use the device in places protected against the weather and far from any sources of heat. After each use, it is recommended to store the device in its own box away from dust and sunlight.
 - In general, it is inadvisable to use single or multiple adapters and/or extensions. Should their use be necessary, you must use ones that are in compliance with safety regulations, however, taking care not to exceed the maximum power supply tolerated, which is indicated on the adapters and extensions.
5. For repairs, exclusively contact SAFETY technical service and request the use of original spare parts. Failure to comply with the above can jeopardise the safety of the device;
6. **This medical device must be destined exclusively for the use for which it has been designed as described in this manual. It must therefore be used as an aerosol therapy system.** Any different use must be considered incorrect and therefore dangerous; the manufacturer cannot be considered liable for damage caused by improper, incorrect and / or unreasonable use or if the appliance is used in electrical plants that are not in compliance with the regulations in force;
7. The medical device requires special precautions regarding electromagnetic compatibility and must be installed and used in accordance with the information provided with the accompanying documents: the PRONTEX WIND device must be installed and used away from mobile and portable RF communication devices (mobile phones, transceivers, etc.) that may interference with the said device.
8. Store the accessories out of reach of children. Children and people with learning difficulties must only use the medical device under the strict supervision of an adult with full mental faculties. Keep the nebulizer out of reach of children under 36 months as it contains small parts that may be swallowed accidentally. **Never leave the device unattended in places accessible to minors and / or the disabled.**
9. Do not leave the device unattended in places accessible to children and / or persons not in full possession of their mental faculties as there is a risk of strangulation with the air tube;
10. The medical device may come into contact with the patient via the nebuliser / masks / mouthpiece and / or nosepiece, components compliant with the requirements of regulation ISO 10993-1: therefore, no allergic reaction and skin irritation may occur.

11. Operation of the device is very simple and therefore no further explanations are required other than those indicated in the following user manual.
12. **WARNING:** None of electric or mechanical parts have been designed to be repaired by customers or end-users. Don't open the device, do not mishandle the electric / mechanical parts. Always contact SAFETY technical assistance.
13. Using the device in environmental conditions different than those indicated in this manual may harm seriously the safety and the technical characteristics of the same;
14. The materials used to contain the drugs are made with highly stable thermoplastic polymers that are resistant against chemicals. Such materials were tested with commonly used drugs (Salbutamol, Beclametasone dipropionate, Acetylcysteine, Budesonide, Ambroxol) and no interaction phenomenon was observed. Interactions cannot however be excluded given the variety and the continuous evolution of the drugs that are used. Remember to:
 - To consume the drugs as quickly as possible after opening its package;
 - To avoid keeping the drug in the tray-like container for too long and to clean it immediately after every application;
 - If the tray-like container presents any abnormal situation (such as softening or cracks), do not introduce any solution and do not proceed with the inhalation. Contact the technical service and describe the methods and type of drugs used.
15. Remember to:
 - Only use this device with medicines prescribed by your doctor;
 - Carry out the treatment only using the accessory indicated by the doctor according to the pathology.
16. Any serious incident occurred in relation to the device should be reported to the manufacturer and the competent authority in your country/area. If you have no contact information of such authority, please contact the manufacturer or distributor whose contact information is indicated in this instruction manual.

The Manufacturer and the Distributor cannot be held liable for accidental or indirect damages should the device be modified, repaired without authorization or should any of its component be damaged due to accident or misuse. Any minimal modification / repair on the device voids the warranty and does not guarantee the compliance with the technical requirements provided by the Regulation 2017/745 and its normatives.

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



Typology (Regulation 2017/745)	Class IIa Medical Device			
Model	PRONTEX WIND			
Power Feeding	230V~ / 50Hz			
Power Consumption	140 VA			
Fuse	F 1 x 1.6A L 250V			
Max Pressure	300 kPa (3.0 Bar)			
Max Air Flow (to compressor)	16 l/min			
Operating Pressure	95 kPa (0.95 Bar)			
Operating Air Flow	5.2 l/min a 95 kPa			
Nebulization (**)	Pos. I (closed) 0,40 ml	Pos. II 0,60 ml	Pos. III 0,70 ml	Pos. IV 0,80 ml
MMAD (µm) * Mass Median Aerodynamic Diameter	Pos. I (closed) 3.32	Pos. II 4.07	Pos. III 4.23	Pos. IV 4.18
GSD (*) Geometric Standard Deviation	Pos. I (closed) 4.12	Pos. II 2.64	Pos. III 2.74	Pos. IV 2.49
Output Rate (ml/min) *	Pos. I (closed) 0.077	Pos. II 0.105	Pos. III 0.124	Pos. IV 0.138
Output (ml) *	Pos. I (closed) 0.189	Pos. II 0.237	Pos. III 0.247	Pos. IV 0.260
Weight	2.00 Kg			
Size	205 (L) x 140 (H) x 150 (P) mm			
Noise Level	60 dB(A)			
Duty Cycle	Continuo			
Min Capacity Nebulizer	2ml			
Max Capacity Nebulizer	8ml			
Working Condition	Room Temperature: 5 ÷ 40°C Room humidity percentage 10 ÷ 93% RH Atmospheric Pressure 700 ÷ 1060 hPa			
Conservation and Transport Condition	Room Temperature: -25 ÷ 70°C Room humidity percentage 0 ÷ 93% RH Atmospheric Pressure 500 ÷ 1060 hPa			

(*) Data determined with Cascade Impactor 290 series, compliant with EN 13544-1, by spraying 2ml of NaF 1.0%

(**) Data from free nebulization of 2ml NaCl 0.9% (average nebulization value per minute)

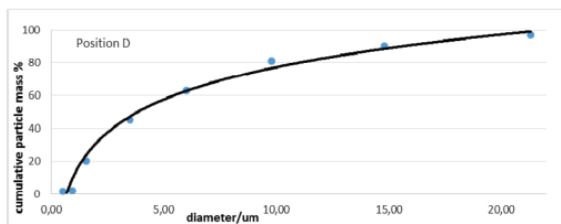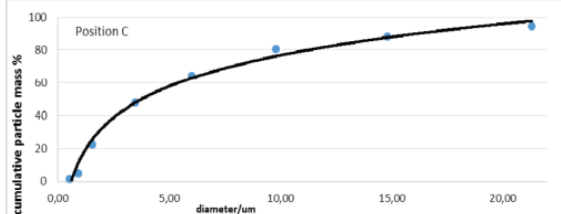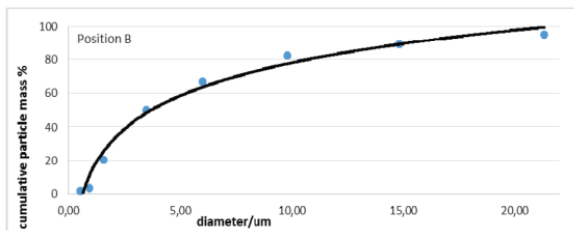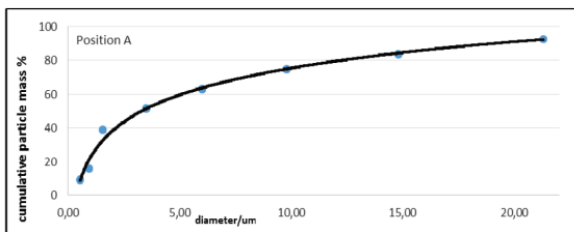NB: The measures and curves are not valid for the high viscosity suspension drug. For more information, consult the medicine manufacturer.

Essential oils of plants, cough syrups, gargling solutions, massage or steam bath oils are absolutely not suitable for inhalation. Their viscosity may impair the operation of the device and consequently the effectiveness of the treatment

IMPORTANT INFORMATION FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EC DIRECTIVE 2012/19/UE-WEEE:

The symbol on the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, don't dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same **Caution:** The wrong disposal of electric and electronic equipment may involve sanctions.

SIMBOLOGY

	Class II isolation equipment	
	CE marking in conformity with Regulation EU 2017/745	
	Medical Device	
	General warnings and/or specifications	
	Consult the instruction manual	
	Applied part type BF (Nebulizer, mouthpiece, nosepiece, pediatric mask and Adult mask)	
	Room Percentage Humidity	
	Operating limit temperature / Transport and storage limit temperature	
	Atmospheric Pressure	
~	Alternate Current	
Hz	Mains Frequency	
I	ON	
0	OFF	
	Lot Number	
	Serial Number	
	Identification device	
	Manufacturer: CA-MI Srl, Via Ugo la Malfa 13 - Frazione Pilastro, 43013 Langhirano (PR) Italy	
IP21	Degree of protection an electrical device provides in the case of accidental or intentional contact with the human body or with objects, and protection in the case of contact with water.	
	1st DIGIT PENETRATION OF SOLIDS	2nd DIGIT PENETRATION OF LIQUIDS
	Protected against solids having a dimension greater than Ø 12mm	Protected against the vertical flow of drops of water

MAINTENANCE

The PRONTEX WIND aerosol does not need maintenance or lubrication.

With regard to training, given the information contained in the user manual and since it is easy to understand the said device, it doesn't appear to be necessary. It is necessary to check functioning and instrument before every use.

Connect cable to electrical network and turn switch on. Close the compressor mouthpiece with a finger and verify that loud noises are not present, these can indicate wrong functioning. Verify that the atomiser is not damaged by previous use (it was badly put away or badly knocked). A protection fuse (**F 1.6 A L 250V**) not reachable from exterior protects the instrument.

For fuse replacing, please make reference to manufacturer technical personnel.

The Manufacturer will provide upon request electric diagrams, components list, description, setting instructions and any other information that can help the technical assistance staff for product repair.

Fault type	Cause	Solution
1. Low Nebulization	Clogged Nebulizer Tank	Clean the nebulizer tank as explained in the instruction manual
2. Low Nebulization	Clogged Nebulizer Tank	If cleaning was not successful change cruet
3. Absence of Nebulization	Clogged Nebulizer Tank	Press the nozzle (cylindrical sleeve) on the inside of the polypropylene nebuliser cup base firmly with one finger
4. Slow Nebulization	Highly dense drug	Dilute drug in physiological liquid The drug must be prescribed by a doctor
5. Noisy Device	Extended use	Call retainer or manufacturer SAFETY
6. The device doesn't work	Power Cable is damaged External power source failure	Replace the power cable Check the external power source
Fault 1 - 2 - 3 - 4 - 5 - 6	No solution with previous items	Call retainer or manufacturer SAFETY

If the unit doesn't nebulizer once the above conditions have been checked, we suggest to contact your dealer or technical service SAFETY.

BEFORE EVERY CHECKING OPERATION, IN CASE OF ANOMALIES OR BAD FUNCTIONING, PLEASE CONTACT SAFETY TECHNICAL SERVICE. SAFETY DOES NOT GIVE GUARANTEE IF INSTRUMENT, AFTER THE TECHNICAL SERVICE CHECKING, APPEARS TO BE TAMPERED.

CLEANING DEVICE

Clean the device and the accessories after each application or before the same, if not properly cleaned. Use a soft dry cloth with not-abrasive and not-solvent detergents. The device's plug must be removed from the wall socket before proceeding with any cleaning procedures.

PARTICULAR CARE SHOULD BE TAKEN TO ENSURE THAT THE INTERNAL PARTS OF THE EQUIPMENT DO NOT GET IN TOUCH WITH LIQUIDS AND THE POWER CORD UNPLUGGED FROM THE SOCKET.

Before storing it and, however, before the next use, wait for the device to be thoroughly dried. Do not expose the compressor to water or heat.

CLEANING AND WASHING OF ACCESSORIES

Before using and/or after cleaning, pay special attention to ensure that all the accessories supplied with the device are intact. Switch off the device before cleaning it and disconnect the mains cable from the electrical socket.

PREPARATION

1. Pull out the air tube from the nebulizer and leave it plugged into the air outlet nozzle of the device;
2. Using your fingers, press the two side tabs of the nebulizer cup that hold the top of the nebulizer cup against the bottom of the same component;
3. Use your fingers to disconnect the internal pisper at the bottom of the nebuliser.

CLEANING

Before and after each use proceed with cleaning all of the components of the nebulizer (with the exception of the air tube) according to one of the two methods described below.

Method 1: Thoroughly clean the components for 5 minutes, using warm drinking tap water (about 40°C) and/or mild soap.

Method 2: Clean the components (except for the air tube) by immersing them in a solution with 60% water and 40% white vinegar. When finished, thoroughly rinse with warm drinking water (approx. 40°C).

After cleaning, rinse thoroughly by removing the excess water and allow to air dry in a clean place.

**DO NOT BOIL OR AUTOCLAVE THE AIR TUBE AND MASKS
DO NOT WASH ACCESSORIES IN A DISHWASHER**

WASHING

If there are pathologies with risks of infection and microbial contamination, it is the end user's responsibility to proceed with suitable washing. The washing procedure can only be carried out if the components to be treated have undergone specific cleaning (see chapter on cleaning). Proceed as follows for the washing procedure:

- Fill a container, of a suitable size to contain all the individual components, with a solution of drinking water and hypochlorite-based solution readily available in a pharmacy, by following the proportions indicated on the packaging of the disinfectant itself;
- The period of time for which is to be immersed in this solution is indicated on the packaging of the hypochlorite solution in accordance with the chosen concentration for preparing the solution;
- Rinse thoroughly with lukewarm drinking water to remove all traces of the solution.
- Dry and store in a dry, dust-free environment.
- Dispose of the used solution according to the instructions provided by the manufacturer of the solution.

STANDARD ACCESSORIES

ACCESSORIES
NEBULIZER KIT (Nebulizer, Connector for MASK, Adult Mask, Pediatric Mask, Air Tube, Mouth-piece and Nosepiece)
Separable Power Cord
Nasal Irrigator

1 Forcella nasale Nosepiece Embout nasal Nasenstück Horquilla nasal	2 Bocheruola Mouthpiece Embout buccal Mundstück Boquilla	3 Ampolla Nebulizer Chambre de nébulisation Jet vernebler Ampolla
4 Tubo aria Air-tube Tube à air Luftschlauch Tubo del aire	5 Maschera pediatrica Pediatric mask Masque pédiatrique Maske für kinder Máscara pediátrica	
6 Connettore Connector Connecteur Stecker Conector	7 Maschera adulto Adult mask Masque pour adulte Maske für erwachsene Máscara para adultos	

For each individual patient it's recommended to use the nebulizer for 6 months or for a maximum of 120 treatments.

The nebulizer must be replaced after a long period of inactivity, if it is deformed or broken, or if the nebulizer nozzle is blocked by dry medicine, dust, etc.. **Only use the original nebulizer supplied by Manufacturer with the device**

Use the "nose piece" accessory only if expressly indicated by your doctor and paying attention **NEVER** to introduce inside the nose the nasal bifurcation, but only bring it as close as possible.

In the presence of infection or microbial contamination prone pathologies, we recommend using your personal accessories and nebulizer (always consult your doctor).

The device is equipped with a filter that removes any impurities from the air that was sucked in by the compressor. The air filter must be replaced every 25 hours of functioning or when it result particularly worn.

For replacement, lift the filter and replace with a new one. Only use original filter.

The mask and tube must be replaced as soon as the materials they are made of show signs of deterioration.


DON'T USE THE DEVICE WITHOUT AIR FILTER


INSTRUCTION FOR USE

- The device must be checked before each use in order to detect malfunctions and / or damage caused by transport and / or storage.
- During the inhalation must sit in an upright and relaxed position at a table and not in an armchair, to avoid compressing the airways and therefore compromising the effectiveness of the treatment.
- It is recommended not to keep the device in your hands and / or to avoid prolonged contact with the body of apparatus.

WARNING: Put the device on a flat and stable surface in order not to block the cooling vents on the sides of the device.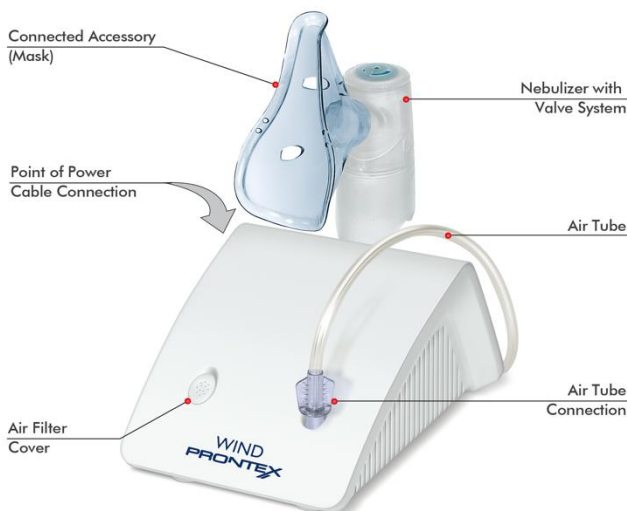

- Use your fingers, press the two side tabs of the nebulizer cup that hold the top of the nebulizer cup against the bottom of the same component;
- Pour the medicine prescribed by the doctor into the nebulizer;
- Re-close the nebulizer;
- Connect air tube to the air exit well positioned on Top Housing;
- Connect the other end of the Air Tube to the connection in the lower part of the nebulizer.;
- Ensure that the supplied air filter is present;
- Connect the desired accessory to the nebulizer (pediatric mask, adult mask, mouthpiece, nosepiece), prescribed by the doctor;
- Connect the power supply cable and insert the plug into the mains socket.
It is recommended to unwind the entire length of the power supply cable to prevent dangerous overheating. If the power supply cable is damaged and must be replaced contact the SAFETY S.p.A. technical service;
- Press switch on position **I** to proceed with nebulization;
- On completing of nebulization, press the switch on position **0** and remove the plug from the socket;
- Wash the nebulizer and its accessories as indicated in the cleaning charter.;
- Place the cable and accessories inside the box.

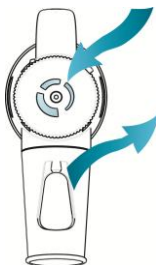Inhalation Valve:

Three vents on top are mechanically opened by patient inhalation force, entraining ambient air. The airflow generated by the compressor along with the airstream entrained by patient enhance the nebulisation activity inside the chamber, thus increasing the volume of aerosol delivered to the patient.

Exhalation Valve:

While remaining closed during inhalation to prevent aerosol to be wasted in the air, the exhalation valve opens when patient starts exhaling, directing the exhaled air in the ambient and preventing mixing with medication in the chamber.

**Speed Selector:**

By rotating and stopping the speed selector on one of the 4 available positions (I-II-III-IV), speed of nebulization can be doubled while maintaining a stable and optimal particle size.

Always use the nebulizer facing upwards so that substances and / or medicines cannot escape from the nebulizer during the normal use.

WARNING: The power supply cable plug is the element of separation from the electrical mains system: even if the units equipped with a special on / off switch button, the power supply plug must be kept accessible once the device is in use so as to allow a further method of disconnection from the mains supply system.

**NEVER INHALE IN HORIZONTAL POSITION
NEVER BEND THE NEBULIZER OVER 60°**

INSTRUCTION FOR USE - NASAL IRRIGATOR

With the device it is disposable a Nasal Irrigator mod. Rhino Care. The Nasal Irrigator is a medical device with Risk Class IIa, which can be connected to active medical device (e.g. Aerosol Therapy Equipment) in Class IIa or higher. Rinho Care is a device used to clean the nasal cavities, hydrates the nasal mucous membranes and generates a micronised nasal spray solution to treat problems of the respiratory tract.

- Lift the adult or child adapter (1) and the collection chamber (2) situated on the main unit (4) of the nasal irrigator;
- Fill the chamber (3) with isotonic and hypertonic solution (max 10ml);
- Put the collection chamber and selected adapter back onto the top of the nasal wash, making sure the parts are perfectly in place;
- Connect the nasal wash (5) to the aerosol therapy device by means of the air tube;
- Place the wash with the selected adapter under a nostril;
- Start the aerosol therapy device (refer to the user manual) and start the wash by pressing and holding the supply button (6);
- During treatment, breathe normally through your nose whilst keeping the other nostril closed;
- Alternate nostrils according to requirement
- When the liquid in the device finishes, or if you want to temporarily stop the treatment, release the nasal wash start button.
- After treatment, empty the collection chamber and clean it.

NOTE: Keep on hand a handkerchief to dry the nostris if mucus is coming out and cannot be collected in the tray on top of the nasal rinsing kit.

CLEANING THE DEVICE - NASAL IRRIGATOR

Due to risk of contamination, empty the device each time it is used

Rhino Care nasal irrigator wash must be cleaned every time it is used. The components must be disassembled as follows:

1. Remove the air connection tube;
2. Remove the adapter (adult or child) and the collection chamber from the top of the nasal irrigator;
3. Remove the green impacter situated inside the nasal wash filling chamber.

The nasal irrigator top and collection chamber are a single piece and cannot be disassembled. Once all components are disassembled, proceed with cleaning: Thoroughly clean the components for 5 minutes, using warm drinking tap water and/or mild soap. After cleaning, rinse thoroughly under running water and removing excess water, and let it dry in a clean place.

Do not use a microware to clean or disinfect the device.

MAINTENANCE - NASAL IRRIGATOR

Cleaning the components of the nasal wash enable you to constantly check their appearance and make it easier to see when they need to be replaced. Rhino Care nasal irrigator must be replaced when signs of deterioration are visible. Rhino Care has no parts that require maintenance. However, it is necessary to perform a few simple checks to verify its functionality before each use.

Problems	Possible Causes	Solution
No nebulised solution comes out from the Rhino Care nasal irrigator	The aerosol therapy device is off	Switch on the aerosol therapy device
	No saline solution in the chamber	Pour saline solution into the chamber (max 10ml)
	Nasal Irrigator start button not pressed	Press and hold the button
	Air Tube not connected correctly or the nasal irrigator was incorrectly assembled	Connect the air tube correctly to the nasal wash connection and air supply connection of the aerosol therapy device. Make sure that all parts of the nasal wash are assembled correctly

If Rhino Care still does not work after the aforesaid conditions have been checked, it is recommended to contact your dealer or SAFETY S.p.A. technical service.

RISK OF ELECTROMAGNETIC INTERFERENCE AND POSSIBLE REMEDIES

This section contains information regarding the conformity of the compliance with the EN 60601-1-2 (2015) +A1 (2021) Standard. PRONTEX WIND is a medical device particularly suitable for home applications. CISPR group and category classification: group 1, category B. PRONTEX WIND is an electromedical device that requires special precautions regarding electromagnetic compatibility and needs to be installed and commissioned according to the information specified in the accompanying documents.

The device should not be used adjacent to or stacked with other equipment, as this could result in improper operation. If such use is necessary and unavoidable, special precautions should be taken so that the electro-medical device functions properly in its intended operating configuration (for example, constantly and visually checking for the absence of anomalies or malfunctions).

The use of accessories, transducers and cables other than those provided by the manufacturer of the device may result in an increase in electromagnetic emissions and/or a decrease in the electromagnetic immunity of this device, causing it to malfunction.

Portable and mobile radio communication devices (mobile phones, transceivers, including peripherals such as antenna cables and external antennas, etc.) may interfere with the medical device and should not be used in close proximity with (at a distance of more than 30 cm from any part of the device, including cables), adjacent to or on top of the medical device. If such use is necessary and unavoidable, special precautions should be taken so that the electro-medical device functions properly in its intended operating configuration (for example, constantly and visually checking for the absence of anomalies or malfunctions).

The following tables supply information regarding the EMC (Electromagnetic Compatibility) characteristics of the electro-medical device.

Guidance and manufacturer's declaration – Electromagnetic Emissions		
The aerosol PRONTEX WIND is intended for use in the electromagnetic environment specified below. The customers or the user of the aerosol PRONTEX WIND should assure that it's used in such an environment.		
Emissions Test	Compliance	Electromagnetic environment - guidance
Irradiated / Conducted emissions CISPR11	Group 1	The aerosol PRONTEX WIND only used RF energy only for its internal functioning. Therefore its RF emissions are very low and are not cause interference in proximity of any Electronic appliances.
Irradiated / Conducted emissions CISPR11	Class [B]	The aerosol PRONTEX WIND can be used in all environments, including domestic and those connected directly to the public mains distribution that supplies power to environments used for domestic scopes or environments feeds to you from batteries.
Harmonic emissions EN 61000-3-2	Class [A]	
Voltage fluctuations / flicker emissions EN 61000-3-3	Complies	

Guidance and manufacturer's declaration – Immunity Emissions			
The aerosol PRONTEX WIND is intended for use in the electromagnetic environment specified below. The customers or the user of the aerosol PRONTEX WIND should assure that it's used in such an environment.			
Immunity Test	Level indicated by the EN 60601-1-2	Compliance Level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) EN 61000-4-2	± 8kV on contact ± 2;4;8;15kV in air	The device doesn't change its state	Floors should be wood, concret or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst EN 61000-4-4	± 2kV power supply lines ± 1kV for input / output lines	The device doesn't change its state	Mains power quality should be that of a typical commercial environment or hospital.
Surge EN 61000-4-5	± 0,5 and ±1kV differential mode	The device doesn't change its state	Mains power quality should be that of a typical commercial environment or hospital.
Loss of voltage, brief voltage interruptions and variations EN 61000-4-11	>95% U _T for 0.5 cycle >95% U _T for 1 cycle 30%U _T for 25/30 cycle (50/60Hz) >95% U _T for 250/300 cycle (50/60Hz)	-	Mains power quality should be that of a typical commercial environment or hospital. If the user of the aerosol PRONTEX WIND request that the appliance operates continuously, the use of a continuity unit is recommended.
Magnetic field EN 61000-4-8	30 A/m	The device doesn't change its state	The power frequency magnetic field should be measured in the intended installation location to assure that it's sufficiently low.
Proximity magnetic field IEC 61000-4-39	30KHz, CW, 8 A/m		
	143.2KHz, PM, 65 A/m		
	13560KHz, PM, 7.5 A/m		
Nota U _T is the value of the power supply voltage			

Guidance and manufacturer's declaration – Immunity Emissions

The aerosol PRONTEX WIND is intended for use in the electromagnetic environment specified below. The customers or the user of the aerosol PRONTEX WIND should assure that it's used in such an environment.

Immunity Test	Level indicated by the EN 60601-1-2	Compliance Level	Electromagnetic environment - guidance
Conducted Immunity EN 61000-4-6	3Vrms 150kHz to 80MHz (for non life-supporting devices)	$V_1 = 3 \text{ V rms}$	The portable and mobile RF communication devices, including cables, must not be used closer to the PRONTEX WIND device, than the separation distance calculated by the equation applicable to the transmitter frequency. Recommended separation distance $d > 30\text{cm}$
Radiated Immunity EN 61000-4-3	10 V/m 80MHz to 2.7 GHz (for non life-supporting devices)	$E_1 = 10 \text{ V / m}$	
Immunity to proximity fields from wireless RF communication devices IEC 61000-4-3	TETRA 400 380 – 390 MHz	27 V/m	$d > 30\text{cm}$
	GMRS 460 FRS 460 430 – 170 MHz	28 V/m	
	LTE Band 13, 17 704 – 787 MHz	9 V/m	
	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5 800 960 MHz	28 V/m	
	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 5 1700 – 1990 MHz	28 V/m	
	Bluetooth, WLAN, 802.11 b/g/n, RIFD 2450, LTE Band 70 2400 – 2570 MHz	28 V/m	
	WLAN 802.11 a/n 5100 – 5800 MHz	9 V/m	

Note 1: At 80 MHz and 800 MHz the interval with the highest frequency is applied

Note 2: These guide lines may not be applicable in all situations. The electro-magnetic propagation is influenced by the absorption and by reflection from buildings, objects and people.

a) The field intensity for fixed transmitters such as the base stations for radiotelephones (mobile and cordless) and terrestrial mobile radio, amateur radio devices, radio AM and FM transmitters and TV transmitters can not be theoretically and accurately foreseen. To establish an electro-magnetic environment generated by fixed RF transmitters, an electro-magnetic study of the site should be considered. If the field intensity measured in the place where the device will be used surpasses the above mentioned applicable level of conformity, the normal functioning of the device should be monitored. If abnormal performance arises, additional measures such as changing the device's direction or positioning may be necessary.

b) The field intensity on an interval frequency of 150 kHz to 80 MHz should be less than 10 V/m.

RULES FOR RETURNING AND REPAIRING

COMPLYING WITH THE NEW EUROPEAN RULES, SAFETY INDICATES THE IMPORTANT POINTS TO PROTECT INSTRUMENT AND OPERATORS HYGIENE. THESE RULES MUST BE RESPECTED IN ORDER TO GUARANTEE HYGIENE AND SAFETY TO ALL THE PEOPLE OPERATING WITH THE INSTRUMENT TO OBTAIN QUALITY AND WELL BEING.

Any device that will be returned to Safety will be checked before repair. If SAFETY finds instrument not suitable for repairing due to clear signs of internal or external contamination, the same will be returned to customer with specification of NOT REPAIRED INSTRUMENT, accompanied by an explanation letter.

SAFETY will decide if contamination is due to bad functioning or misuse.

SAFETY is not responsible for contaminated accessories, they will be substitute at customer's expenses.

For this reason it is **COMPULSORY** to carefully disinfect the external part of the instrument and accessories with a cloth soaked in methylated spirits or hypochlorite-based solutions.

For disinfection of the accessories it is recommended to place them in the same disinfectant solutions for a period reported on the packaging of the hypochlorite solution according to the concentration chosen for the preparation of the solution.

Put the device and accessories in a bag with indication "Device and Accessories disinfecting".

We also request to specify the kind of fault, in order to speed up repairing procedures.

It is required to carefully read the instruction for use to avoid compromising the device with inadequate use.

Always specify the fault encountered so that SAFETY can establish whether it falls into the category of the faults covered by the guarantee.

SAFETY warrants its product for **5 years** after purchasing date against any material or construction defects, provided that no intervention or tampering has been carried out on the device.

A Replacement will be provided only if a SALES RECEIPT and STAMPED GUARANTEE are provided.

Warranty is defined as replacement or repair of component's device that are defective at the origin due to manufacturing defects.

The Warranty does not cover:

- The included accessories and parts subject to normal wear
- Operations on presumed defects or for convenience
- Operations for break downs or defects due to improper use, neglect or negligence and if the damage is not imputable to the manufacturer (accidental dropping, careless transportation, etc).

This warranty does not imply any reimbursement for direct or indirect damages of any nature to persons, animals or objects stemming from the improper use of the product or its period of inefficiency.

IN COMPLIANCE WITH THE ABOVE CONDITIONS AND TO APPLY FOR WARRANTY, PLEASE SEND THE DEVICE ALONG WITH AND THE PURCHASE RECEIPT, TO:

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Safety SpA – Via G. Di Vittorio, 17 - 20813 Bovisio Masciago (MB).
Safety S.p.A.